STARFIT™

ENTERAL DRAINAGE BAG SYSTEM

Single-use device, to be replaced every 24 hours.

Does not contain LATEX or DEHP or BPA.

Store dry and protected from light.

Manufacturer :

HOIST MEDICAL

23 Quai de Rive Neuve

13007 Marseille France

Tel : + 33 (0) 4 91 76 73 29

www.hoist-medical.com

Indications

The STARFIT™ enteral drainage system with ENFit® connector is a closed system for the drainage and collection of gastrointestinal tract contents in newborn, paediatric and adult patients.

Precautions and warnings

Do not use if package has been opened or damaged.

Check the expiry date and do not use after this date.

To be used only with enteral feeding devices with ENFit® connectors and ONLY FOR ENTERAL DRAINAGE.

Care must be taken to always keep the bag upright to avoid wetting the antibacterial filter, which will affect its performance.


To prevent contamination, the enteral drainage bag should not be placed directly on the ground.

To prevent reflux of gastric contents, avoid compressing the drainage bag during use.

Device to be destroyed after use according to the facility's biological waste management protocol. For home use, health care workers will take care of waste management. Do not dispose of the device in household waste.

Description

The STARFIT™ enteral drainage bag system consists of a bag available in 4 sizes: 100 mL; 250 mL; 500 mL and 1000 mL.

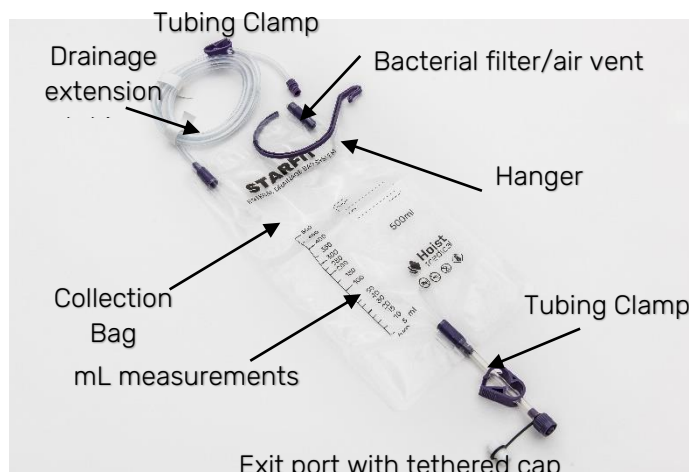The drainage tubing for connection to the gastric tube equipped with an ENFit® connection measures 75cm for sizes 100 and 250 mL and 150cm for sizes 500 and 1000 mL.

The system is equipped with an antibacterial filter that evacuates/vents gases from the bag.

The front of the bag is graduated with mL measurements to visualize the progress of drainage (these graduations are approximate).

A location allows, by means of an indelible marker, to record information about the patient and the date of application, directly on the bag.

At the bottom of the bag is the drain tubing consisting of a large bore section tube, a clamp and an ENFit® connector in accordance with ISO 8036 9-3.

A purple hanger, included, allows the bag to be suspended from the patient's bed or chair.

Instructions for use

Wash your hands before handling the device and its components and follow the hygiene rules established by the establishment.

Close the clamp, as shown (Fig. 1).

Connect the end of the drainage bag connector to the connector of the feeding tube (Fig. 2).

Fig. 1

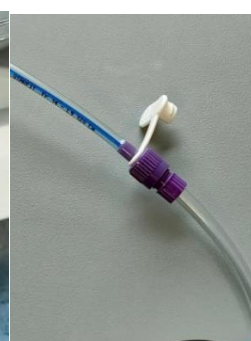
Fig. 2

Notes :

Fig. 3

[illegible][illegible]

Fig. 4

[illegible][illegible]

-
- This image shows a vertical rectangular sheet of white paper. It features approximately 20 evenly spaced horizontal dashed lines, which are typical for primary school handwriting practice. The lines extend across the full width of the page. There are no margins, text, or other markings on the paper.

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