


STOMAFIT™

STOMA STOPPER

Single-use device
Does not contain LATEX or DEHP or BPA.
Store dry and protected from light.

Manufacturer :
HOIST MEDICAL
23 Quai de Rive Neuve
13007 Marseille France
Tel : + 33 (0) 4 91 76 73 29
hoist-medical.com

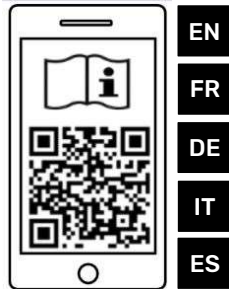

Indications

If a feeding tube is accidentally dislodged or removed, it may only take 1 - 2 hours for the stoma tract to close. It is therefore critical to maintain stoma tract patency and to avoid additional invasive procedures to either dilate or create a new stoma, that the tracts integrity is maintained

The STOMAFIT stoma stopper is intended for pediatric and adult patients to prevent stoma closure if their stoma device has been displaced unintentionally or between device changes

Precautions and warnings

- Do not use if package has been opened or damaged.
- Check the expiry date and do not use after this date.
- To be used only with well-established stoma.
- Device to be destroyed after use according to the facility's biological waste management protocol.
- For home use, health care workers will take care of waste management. Do not dispose of the device in household waste.

Description

The STOMAFIT is a stoma stopper made from Polyurethane.

The STOMAFIT is available in several French Sizes (08Fr to 20Fr) and several lengths (15 mm, 30 mm, 60 mm and 100 mm) to accommodate all stoma sizes.

STOMAFIT discs are color coded according to the French size as below:

French Size	Color Code
08 Fr	Blue
10 Fr	Black
12 Fr	White
14 Fr	Green
16 Fr	Orange
18 Fr	Red
20 Fr	Yellow



Instructions for use

- Wash your hands before handling the device and its components and follow the hygiene rules established by the facility.
- Select the adequate French Size and Length of the Stopper according to the removed or dislodged device
- Gently insert the Stopper into the stoma, if needed lightly lubricate the stopper extremity
- Cover the Stopper with the accompanying dressing