STARFIT™

ENTERAL DRAINAGE BAG SYSTEM

Single-use device, to be replaced every 24 hours.
Does not contain LATEX or DEHP or BPA.
Store dry and protected from light.

Manufacturer :
HOIST MEDICAL
23 Quai de Rive Neuve
13007 Marseille France
Tel : + 33 (0) 4 91 76 73 29
hoist-medical.com

Indications

Indications

The STARFIT™ enteral drainage system with ENFit® connector is a closed system for the drainage and collection of gastrointestinal tract contents in newborn, paediatric and adult patients.

Precautions and warnings

Do not use if package has been opened or damaged.
Check the expiry date and do not use after this date.
To be used only with enteral feeding devices with ENFit® connectors and ONLY FOR ENTERAL DRAINAGE.

Care must be taken to always keep the bag upright to avoid wetting the antibacterial filter, which will affect its performance.


To prevent contamination, the enteral drainage bag should not be placed directly on the ground.

To prevent reflux of gastric contents, avoid compressing the drainage bag during use.

Device to be destroyed after use according to the facility's biological waste management protocol. For home use, health care workers will take care of waste management. Do not dispose of the device in household waste.

Description

The STARFIT™ enteral drainage bag system consists of a bag available in 4 sizes: 100 mL; 250 mL; 500 mL and 1000 mL.

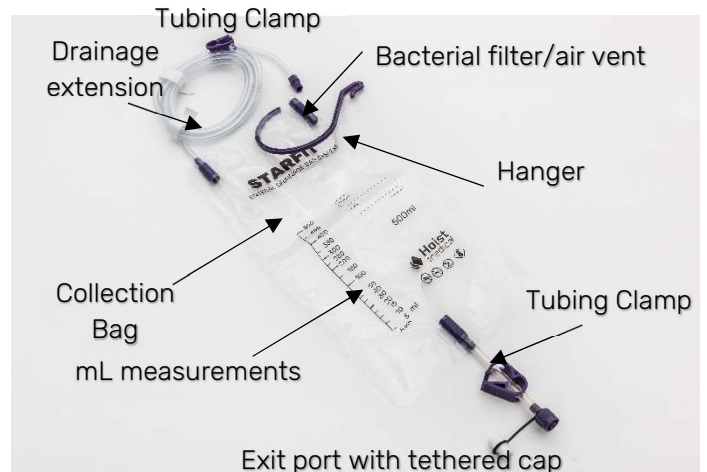The drainage tubing for connection to the gastric tube equipped with an ENFit® connection measures 75cm for sizes 100 and 250 mL and 150cm for sizes 500 and 1000 mL.

The system is equipped with an antibacterial filter that evacuates/vents gases from the bag.

The front of the bag is graduated with mL measurements to visualize the progress of drainage (these graduations are approximate).

A location allows, by means of an indelible marker, to record information about the patient and the date of application, directly on the bag.

At the bottom of the bag is the drain tubing consisting of a large bore section tube, a clamp and an ENFit® connector in accordance with ISO 80369-3.

A purple hanger, included, allows the bag to be suspended from the patient's bed or chair.

Instructions for use

Wash your hands before handling the device and its components and follow the hygiene rules established by the facility.

Close the clamp, as shown (Fig. 1).

Connect the end of the drainage bag connector to the connector of the feeding tube (Fig. 2).

Fig. 1

Fig. 2

Place the drainage bag vertically and secure it under the patient's stomach. Use the hanger provided (Fig.3).

Fig. 3

The drainage tubing must be secured to ensure that the liquid is discharged to the drainage bag. The excess tube may be wound but positioned between the patient's stomach and the bag.

Drainage can be initiated by connecting an ENFit® enteral syringe to the system's drain connector and vacuuming with a syringe to initiate drainage. (Fig. 4).

Fig. 4

Inspect the system regularly to ensure that fluid flows freely into the bag and is not clogged. Empty the bag at least twice a day or when it is half or 3/4 full.

To do this:

- Close the clamp of the tubing.
- Gently disconnect the feeding tube and close the tube cap.
- Empty the bag above the disposal receptacle (toilet or drainage container) and open the drain clamp. Caution: To prevent contamination, the drainage hole should not touch the container or toilet.
- After emptying the bag, clean the drainage exit port with alcohol.
- Close the clamp of drain tubing.

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