# PURFIT™

## GASTROSTOMY BUTTON PURFIT™

Single use device.  
 Replace the button every 3 to 6 months.  
 Replace extension set weekly.  
 Sterile device (EO).  
 Do not reuse, do not resterilize.  
 Does not contain LATEX nor DEHP or BPA.  
 Store dry and protected from light.

Manufacturer :  
**HOIST MEDICAL**  
 23 B Quai de Rive Neuve  
 13007 Marseille France  
 Tel : + 33 (0) 4 91 76 73 29  
 hoist-medical.com

EN  
FR  
DE  
IT  
ES

### Indications

### Button

A medical device made from PUR that allows fluid feeds to be given directly into the stomach for long-term feeding for patients unable to tolerate oral feeding. It is held in place by an internal balloon filled with air or water. The button is placed to replace the initial gastrostomy tube after the stoma is well established.

### Accessories

The extension is used to connect the button to the food container  
 The 10 ml syringe is used to inflate the retention balloon of the button  
 The 60 ml syringe is used to deliver enteral nutrition through the extension  
 The Lubricant is used to lubricate the button for an easy insertion through the stoma  
 The compress is used to clean the button and the stoma

### Contraindications

Ascites, Colon interposition, Portal hypertension, Peritonitis, Morbid obesity

### Description


The PURFIT™ low profile gastrostomy button is a sterile device made of POLYURETHANE consisting of a button with a tube (radiopaque line throughout) and internal retention balloon.

Available sizes are: 12 Fr; 14 Fr; 16 Fr; 18 Fr; 20 Fr; 22 Fr; 24 Fr.  
 The available lengths are: 0.6cm; 1.0cm, 1.3cm; 1.6cm; 2.0cm; 2.3cm; 2.6cm; 3.0cm; 3.5cm; 4.0cm; 4.5cm; 5.0cm.


Size and length are engraved unalterably on the purple nutrition port of the device, the lot number is engraved on the cap tab.

The button has two ports: the anti IV nutrition port is located in the center of the purple disc,( color coded for enteral ) it contains a non-return valve and serves as a connection with the nutrition tubing called the extension set. This extension set consists of a 30cm tube (60cm available as an option), with clamp, an ENFit® connector in accordance with ISO 80369-3 with tethered cap and a CLICK & CONNECT® connector for connection to the button's nutrition port.

Please note: extensions should not be reused for more than one week.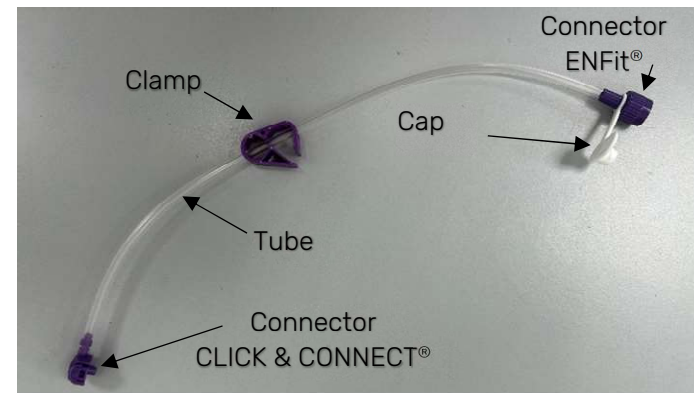


The side port marked **INF** (for inflation) is the balloon inflation port. Operated by a Luer Slip syringe, it is equipped with a non-return valve. The balloon volume is printed on the inflation port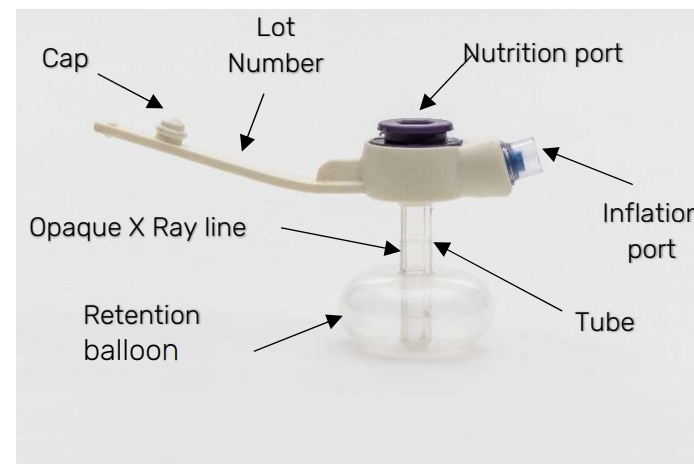

The packaging of the device includes:

- A single box made of recyclable cardboard.
- A booklet for the patient/carer and instructions for use for healthcare professionals.
- A peelable tray with a TYVEK® seal that contains:
  - A PURFIT™ low profile gastrostomy button.

- Two 30cm PURFIT™ extension sets, one with single connector ENFit® and one with Y ENFit® connector with tethered caps.
- 10mL Luer Slip syringe for inflating the retention balloon.
- 60 mL enteral ENFit® syringe for nutrition, medication administration or flushing.
- Aqueous gel for lubrication. (on insertion)
- 4 gauze pads.
- 2 identification labels for the patient's medical records and notes.

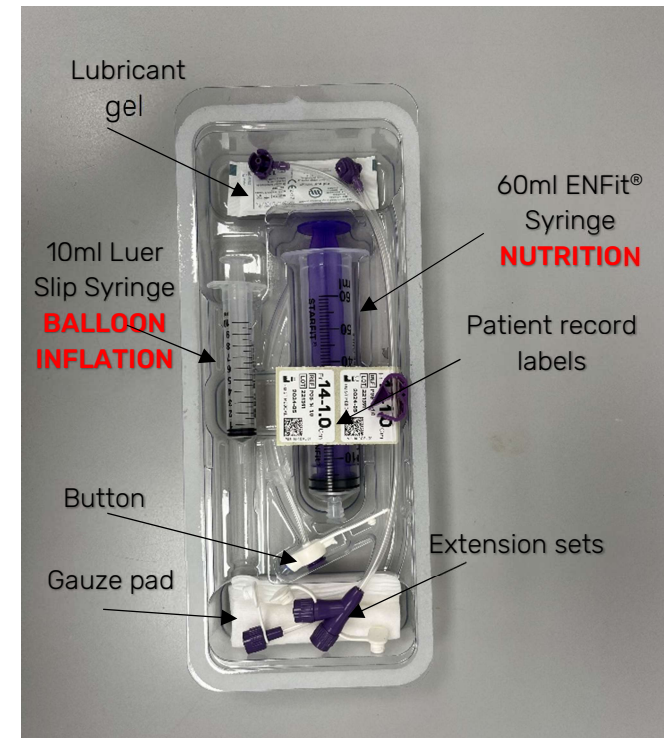### Precautions and warnings

To be used by trained medical staff, trained patient or trained patient family.  
 Do not use if package has been opened or damaged.  
 Check the expiry date and do not use after this date.  
 The placement of the PURFIT™ low profile gastrostomy button should be performed by, or under the direction of, a physician.  
 Sizes (Fr) and length (cm) are prescribed by the physician.  
 Single use button, do not resterilise or retreat. Reuse may create a risk of infection and/or compromise the reliability of the device.  
 Do not cut, tear, pierce, twist, knot, or perform any other action that could damage the gastrostomy button or extension set.  
 The PURFIT™ low profile gastrostomy button can remain in a well-established gastrostomy for 3 to 6 months..  
 The lifespan of a PURFIT™ low profile gastrostomy button may vary depending on several factors including, but not limited to balloon inflation volume, gastric pH, after care and maintenance.  
 Always keep a replacement PURFIT™ low profile gastrostomy button (same size and length), a gastrostomy tube or any device to prevent closure of the stoma.  
 Complications when using a gastrostomy button, but not limited to, are:

Infection, intraperitoneal leak, mucosa, mycosis, pressure necrosis, hyper-granulation, skin lesion.

### Initial placement and replacement

Select the size and length of the button as prescribed by the physician or according to the device previously used. (Using the right size helps to prevent gastric leakage or skin irritation. In case of initial placement, the physician will have used a stoma measuring device to measure the stoma length). A button that is too long can cause gastric leakage and a button that is too short can cause necrosis by pressure. If in doubt, always opt for the longer, adjusting the volume of water in the balloon to reduce movement.

Wash hands before handling the device and its components.

Removal of the initial device:

Ensure the device to be removed is free moving by rotating a full turn.

Completely deflate the balloon with a 10 mL Luer Slip syringe. Again gently rotate the device to be replaced and then pull very carefully, in one swift movement to remove it. If there is resistance, try deflating the balloon again to ensure there is no residual in the balloon and try again. If there is still resistance, notify a physician immediately.

Once removed, clean the skin around the stoma with a wet gauze in gentle circular motions from the center of the stoma to the periphery.

Check the integrity of the device before use.

Check the uniformity of the balloon by filling it with 4 mL of air using the 10 mL Luer Slip syringe supplied in the package. The inflated balloon should be concentric and have a regular shape. Wait a few minutes to check that it does not deflate.

Crushing the balloon could damage it. After checking, deflate the balloon completely and lubricate the end of the device with the water-based lubricating gel provided in the package. Do not use oil or petroleum jelly.

Gently guide the tube through the stoma into the stomach, until the head of the PURFIT™ low profile gastrostomy button sits on the patient's skin.

Keep the device in contact with the skin with gentle pressure and inflate the balloon to the recommended **Volume 4 mL for Fr 12 Fr 14 Fr 16 mL Volume 6 ml for Fr 18 Fr 20 Fr 22 Fr 24** using the 10 mL Luer Slip syringe provided in the package.

The balloon can be inflated with water or air. Inflation with air is recommended to space out checking and reinflation operations.

Check for signs of gastric leakage.

Use a gauze compress to clean the stoma site and to remove any excess lubricant.

Before you commence feeding, you should check its positioning using the extension.

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