

 **STOMAFIT™**
STOMA STOPPER

Single-use device

Does not contain LATEX or DEHP or BPA.

Store dry and protected from light.

Manufacturer :

HOIST MEDICAL

23 Quai de Rive Neuve

13007 Marseille France

Tel : + 33 (0) 4 91 76 73 29

www.hoist-medical.com



Indications

It may only take 1 - 2 hours for a stoma tract to close. It is therefore critical to maintain stoma tract patency and to avoid additional invasive procedures to either dilate or create a new stoma, that the tracts integrity is maintained.

The STOMAFIT™ Stoma Stopper is intended for pediatric and adult patients to prevent stoma closure and maintain stoma integrity if the stoma device has been displaced unintentionally or between device changes

Precautions and warnings

- Do not use if package has been opened or damaged.

- Check the expiry date and do not use after this date.

- To be used only with well-established stoma.

- Device to be destroyed after use according to the facility's biological waste management protocol.

- For home use, health care workers will take care of waste management. Do not dispose of the device in household waste.

Description

The STOMAFIT™ is a stoma stopper made from Polyurethane.

The STOMAFIT™ is available in several French Sizes (08Fr to 20Fr) and several lengths (15 mm, 30 mm, 60 mm and 100 mm) to accommodate all stoma sizes.

STOMAFIT™ discs are color coded according to the French size as below:

French Size	Color Code
08 Fr	Blue
10 Fr	Black
12 Fr	White
14 Fr	Green
16 Fr	Orange
18 Fr	Red
20 Fr	Yellow



Instructions for use

- Wash your hands before handling the device and its components and follow the facility's hygiene rules.

- Select the appropriate French Size and Length of the STOMAFIT™ Stoma Stopper according to the removed or dislodged device

- Gently insert the STOMAFIT™ Stoma Stopper into the stoma, if needed lubricate the Stopper shaft with waterbased lubricate.

- Cover the STOMAFIT™ Stoma Stopper with the dressing provided.