STARFIT™

ADULT NASOGASTRIC FEEDING TUBE

Single-use device.

Length of Use up to 30 days.

Sterile device(EO).

Do not reuse, do not resterilize.

Does not contain Latex or DEHP or BPA.

Store dry and protected from light.

Manufacturer :

HOIST MEDICAL

23 Quai de Rive Neuve

13007 Marseille France

Tel : + 33 (0) 4 91 76 73 29

hoist-medical.com

EN
FR
DE
IT
ES
SE

Indications

STARFIT™ tubes are designed for nasogastric enteral feeding, passive gastric drainage/aspiration or nasointestinal feeding in patients requiring enteral nutrition or NG/Nl access.

Contraindications

Organic intestinal occlusion.

Repeated or incoercible vomiting.

High flow fistulas.

Severe acute anchoratitis in the first 8 days.

Severe diarrheas.

Insufficient effective intestinal absorbent surface.

Description


The tubes are made of medical grade polyurethane with a 40% barium radiopaque line.

The NGG, NGW, NGGV, NGGBY and NGWBY references are equipped with a stainless steel stylet that facilitates insertion.

Weighted NGW and NGWBY tubes have a distal end weighted with tungsten.

STARFIT™ tubes are available in size Fr 06 Fr 08; Fr 10; Fr 12 and Fr 14 (Fr 16 and Fr 18 on request). The length is 80 cm 92 cm 120cm 140 cm.

The numeric centimetric markings printed on the tube facilitate insertion, documentation and allow for migration monitoring. All feeding tubes have ENFit® connector in accordance with ISO8036 9-3.

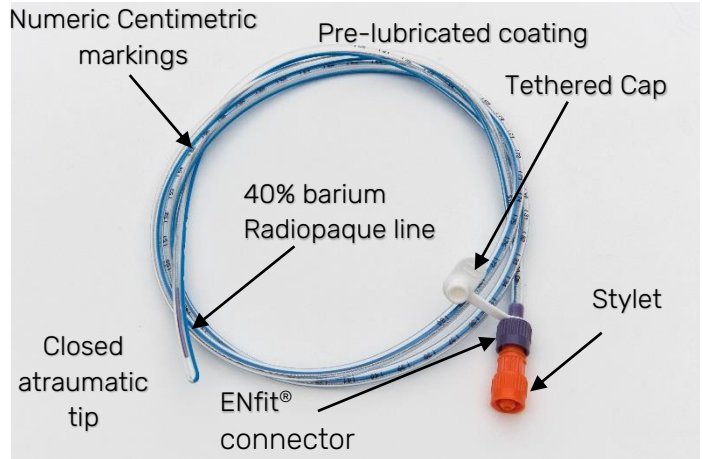Precautions and warnings

To be used by trained medical staff, trained patients or trained patient family.

Do not use if package is opened or damaged.

Check the expiry date and do not use after this date.

If lubrication is deemed necessary, use water based lubricant, do not use mineral oil or petroleum jelly.

Always check the position of the tube:

By aspiration of gastric fluid (with a 50 or 60 mL ENFit® enteral feeding syringe).

The position of the feeding tube on high-risk patients (comatose patients, non-reactive patients, etc.) may be confirmed by radiology.

Insertion of the tube into the trachea, bronchi or lungs creates serious risks for the patient

Check that the tube has not wrapped into the patient's back of the throat. Never reintroduce the stylet when the tube is already in place.

Check the tube position periodically and systematically before each administration. Document the cm marking at patients' nares to observe for signs of migration.

The use of a syringe with a volume of less than 50 mL for positioning verification and flushing should be used with care as may generate pressures greater than the tube rupture pressure. Small volume syringes 1 mL to 20 mL are advised for drug administration only, with gentle pressure.

The tube should be flushed with 20 to 30 mL of water (in a 50 or 60 mL ENFit® syringe) whenever feeding is interrupted and systematically every 6 hours.

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