PURFIT™

STOMA MEASURING DEVICE

Single-use device,

Duration of use - Temporary (less than 60 minutes).

Contains no Latex, DEHP or BPA.

Store dry and away from light.

Manufacturer :

HOIST MEDICAL

23 Quai de Rive Neuve

13007 Marseille France

Tel : + 33 (0) 4 91 76 73 29

hoist-medical.com

EN

FR

Indications

The STOMA MEASURING DEVICE is intended to be used to determine the depth (length) of an established stoma tract, in order to select the appropriate length of the **PURFIT™** balloon gastrostomy button.

Precautions and warnings

To be used by qualified medical personnel,

The length and diameter of the button should be determined by a healthcare professional.

Use only for an established stoma.

Do not use if the packaging is opened or damaged. Check the expiration date and do not use after this date.

If lubrication is required, on the advice of the doctor, only use water based lubricant - do not use mineral oil or petroleum jelly

Description

The **PURFIT™ STOMA MEASUREMENT DEVICE** contents, comprises of a measuring tool with flexible shaft, printed graduated numeric scale and a retention balloon. A balloon inflation valve is situated at the proximal end.

A 5 ml Luer Slip syringe for balloon inflation.

A sachet of water-soluble lubricating gel to lubricate the end of the tube before use

Device to be destroyed after use, according to the establishment's biological waste management protocol. For home use, the healthcare provider will provide clinical waste management. Do not dispose of the device in household waste.

Instructions for use

Please note: The following instructions are given for information purposes only, certain conditions may require modifications.

- Lubricate the end of the tube with the lubricating gel. Do not use mineral oil or petroleum derivatives
 - Gently insert the end of the device through the stoma into the stomach for at least 6 cm.
 - Using the luer slip syringe draw up 4 ml of water or air and connect to the inflation valve - inflate the balloon.
 - Pull the device gently outward until resistance is felt - this indicates the balloon is in contact with the stomach wall
 - Slide the external disc down until it touches the patient's skin.
 - Read the graduation marking above the disc;
 - If possible, it is preferable to take two measurements: one with the patient in a standing position and the other in a supine position. Calculate the average of the two measurements and select the larger size.
 - Deflate the balloon with the luer slip syringe.
 - Carefully remove the measuring device.
-

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